
Instructions for use TOPRO Taurus H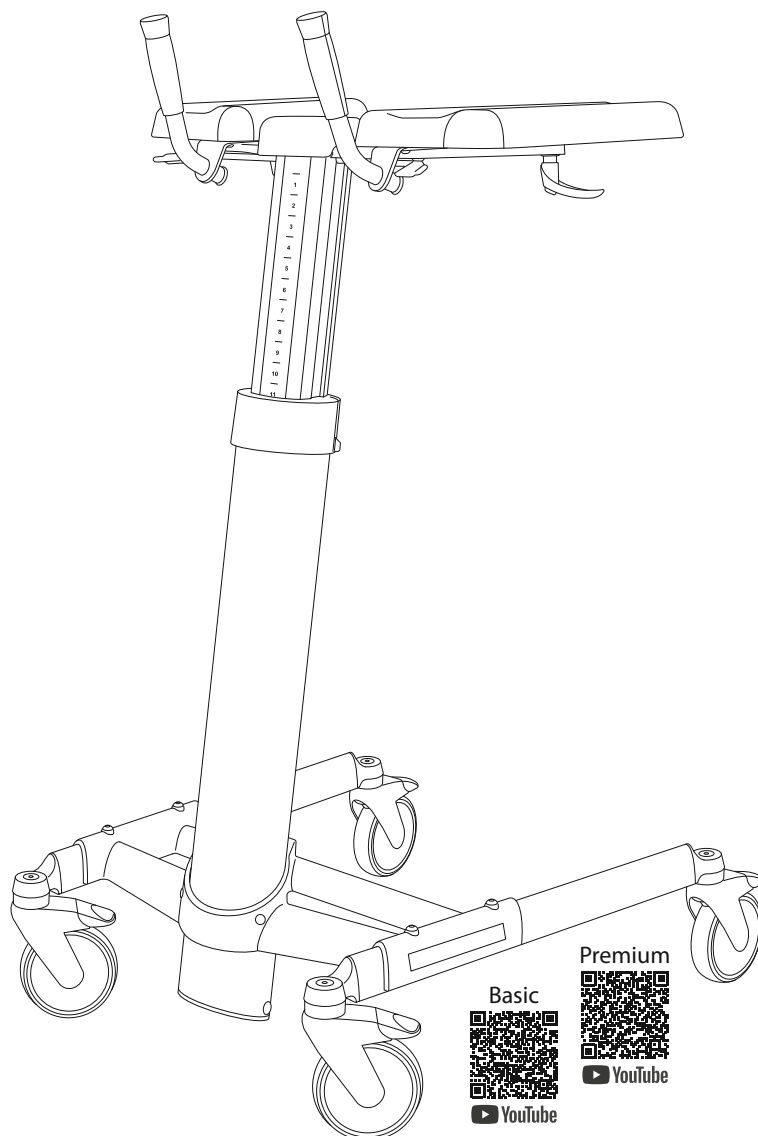

topromobility.com

made in Norway

EN

NO

DE

NL

SV

DA

FI

IS

FR

IT

ES

CS

EN

Illustrations
Dimensions
Technical data
Article numbers

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




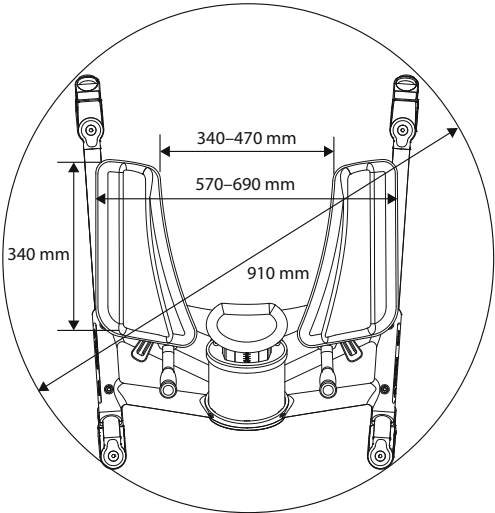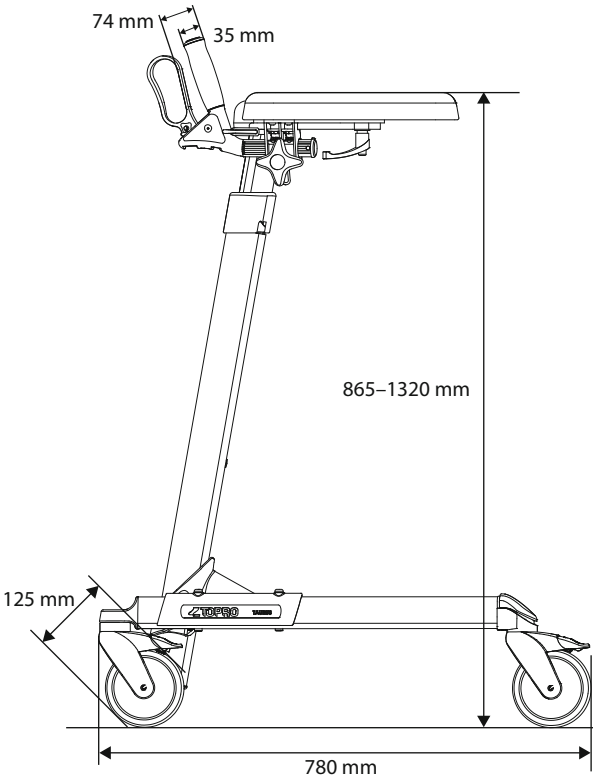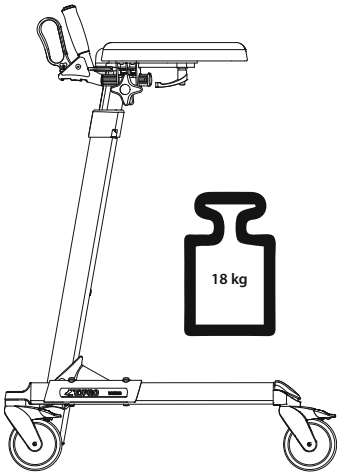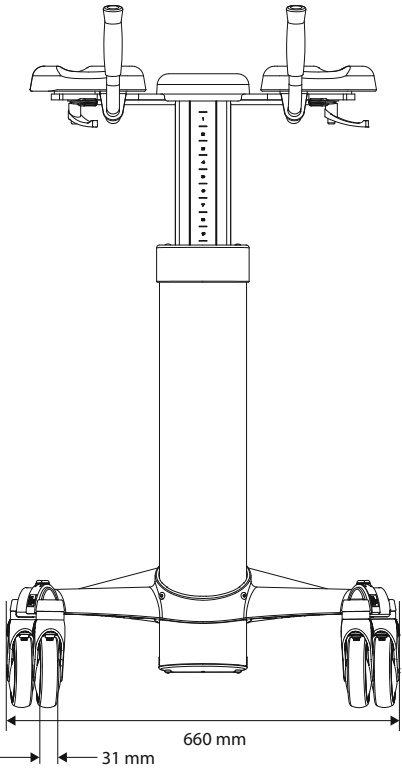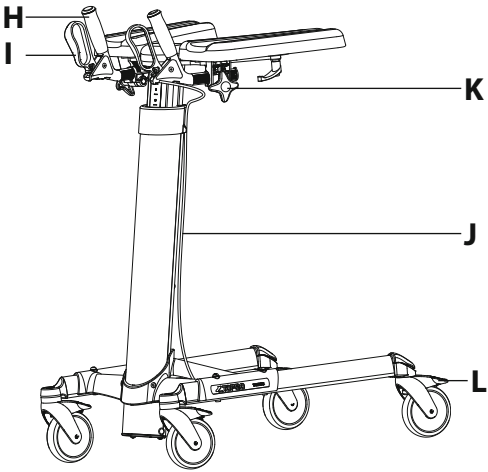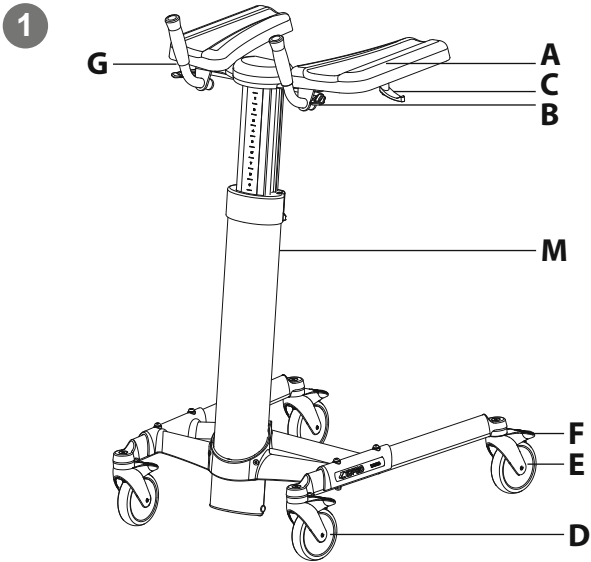


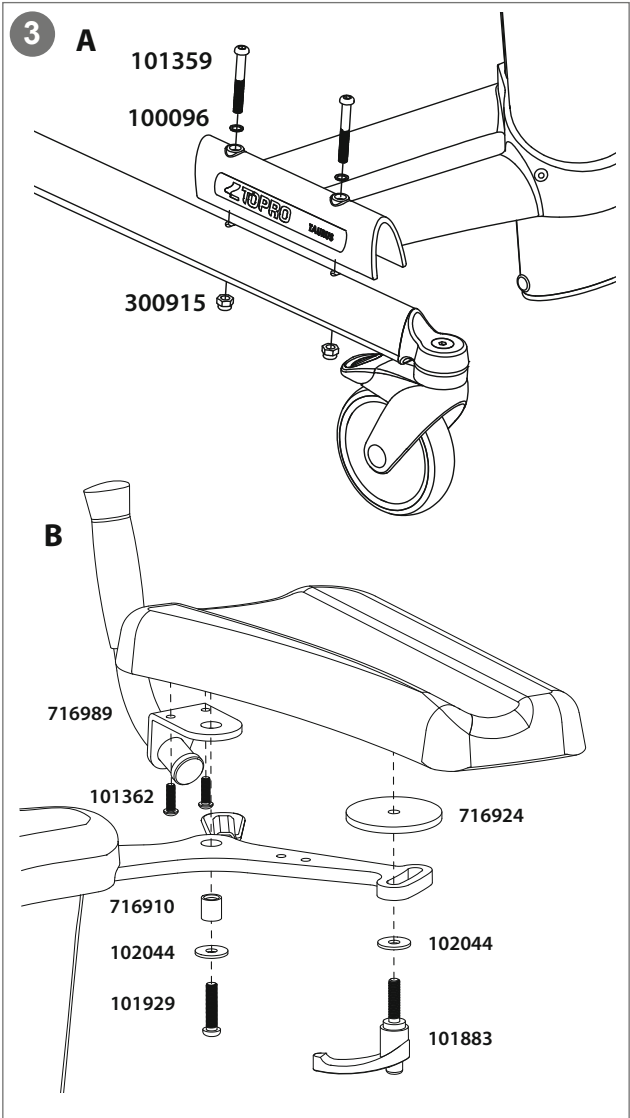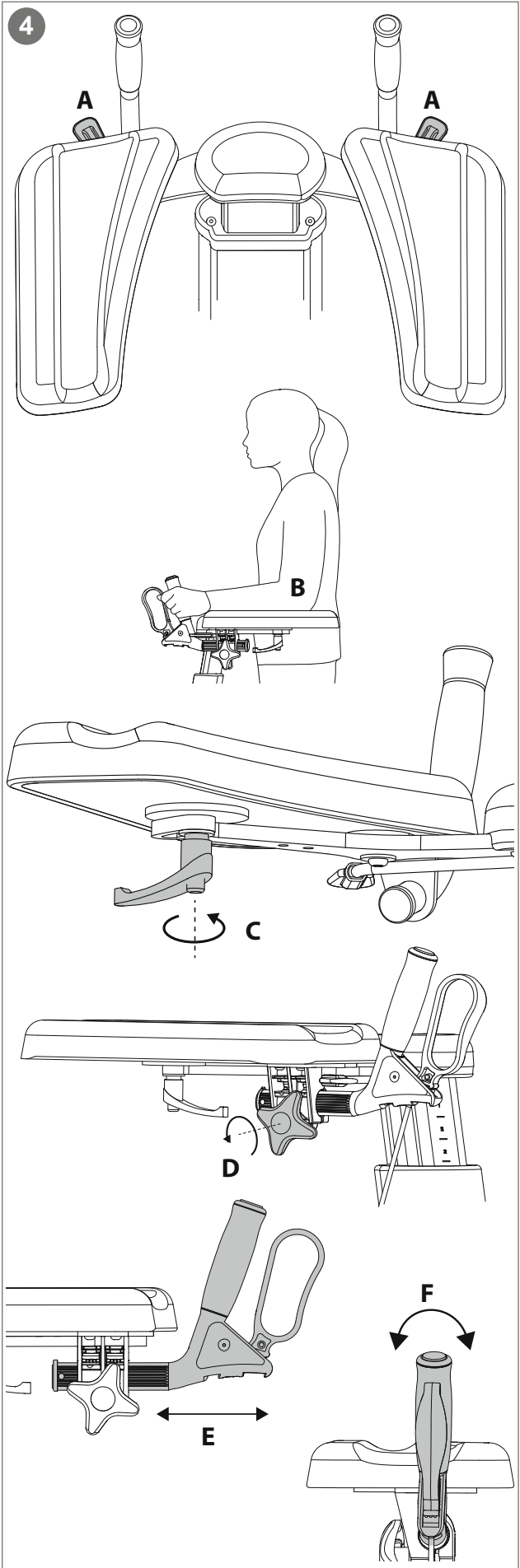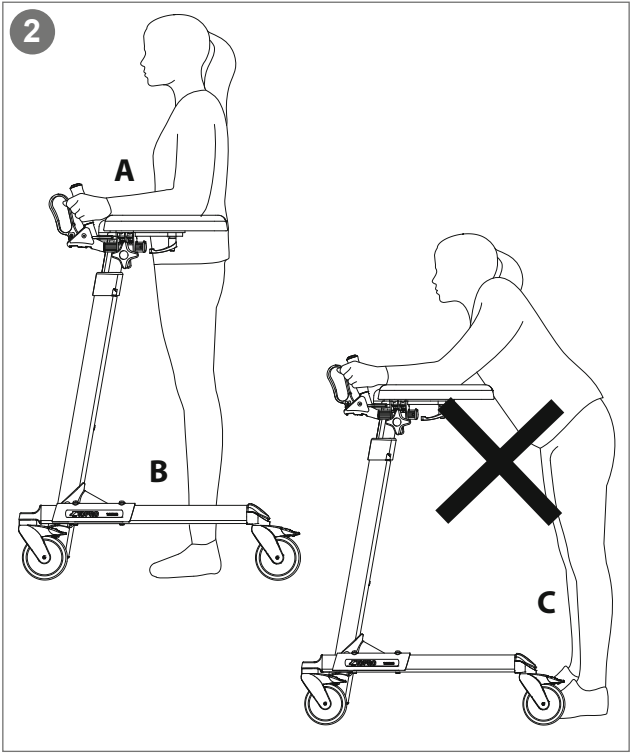

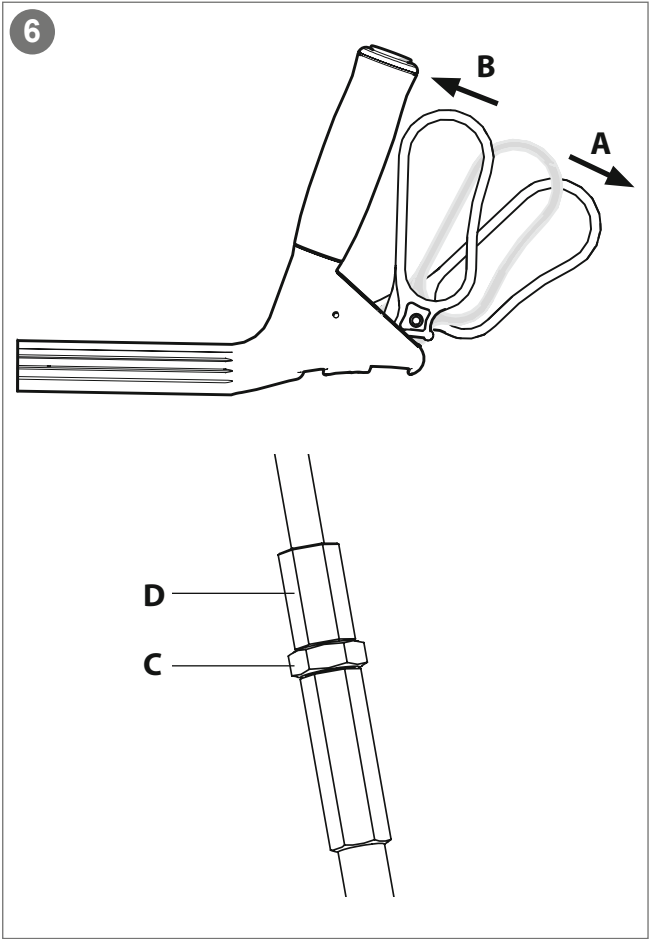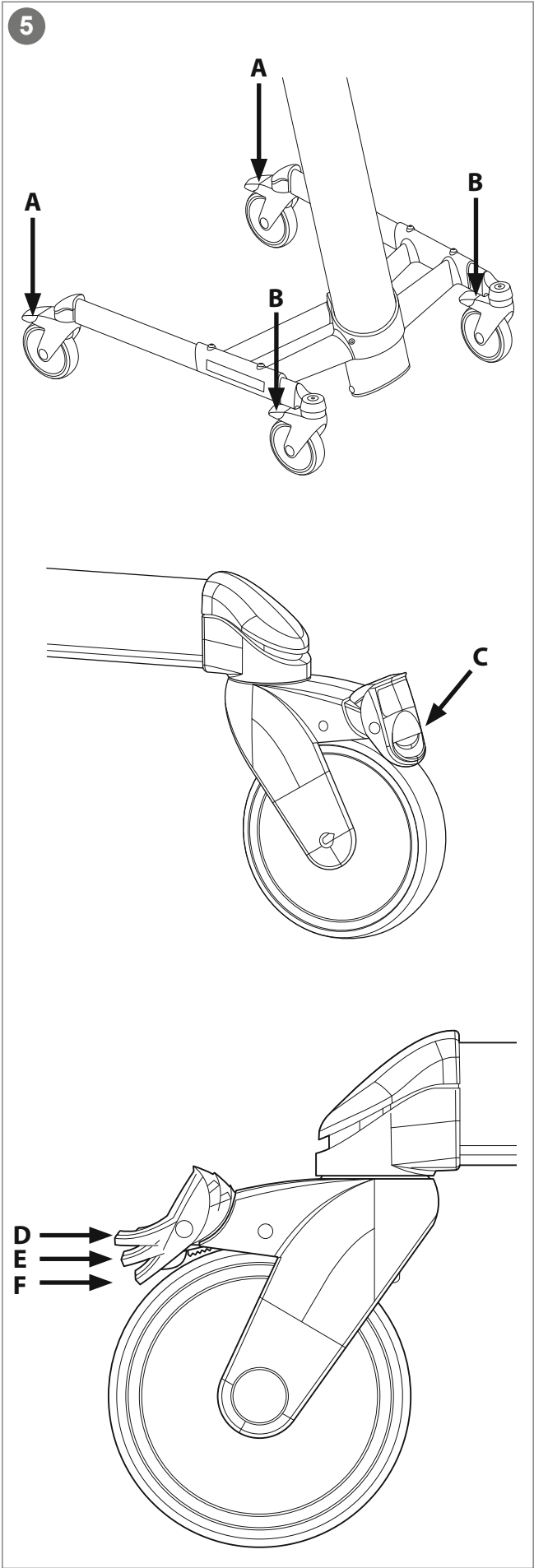

11

12

13 TOPRO Industri AS
Rambekkvegen 5,
NO-2816 Gjøvik, Norway
www.topromobility.com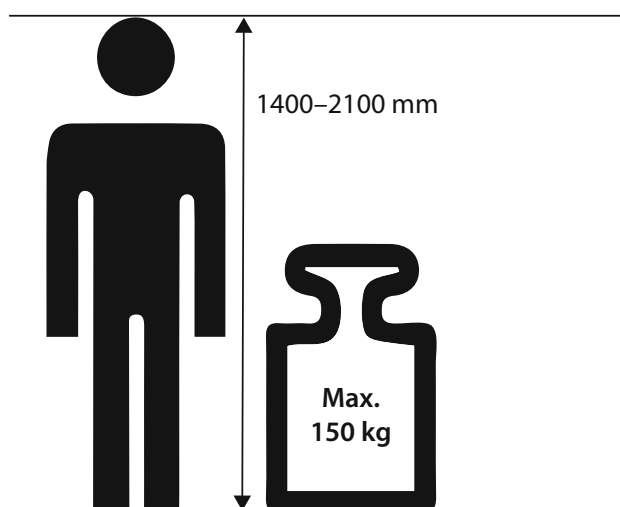

Congratulations with your choice of a new TOPRO forearm walker

Every TOPRO forearm walker is developed, designed and manufactured in Norway. The product conforms to Medical Device Regulation (EU) 2017/745. It is tested and approved according to EN ISO 11199-3:2005. On receipt of the goods, please check your product. If you have any queries, please contact your mobility shop or dealer immediately. If you find it challenging to read this user manual, an electronic version is available on topromobility.com.

Intended use

The TOPRO Taurus H is a forearm walker, which shall give support to users with reduced balance and/or reduced walking ability. The forearm walker is designed to be pushed, not pulled. The intended user is an adult. Height and weight restrictions apply. The main user of the product is the patient him-/herself; however, the walker can also be operated by an assistant. *Contraindications: The product is not suitable for persons with low strength in arms and legs, with very poor balance, or with considerable cognitive disabilities. The product is designed and approved for indoor use on plain foundation. We advise caution when using the TOPRO Taurus in private homes with narrow doorways, as the maximum width of the walker is 660 mm.*

To get to know the forearm walker and learn how to use it properly, training by a professional is recommended.

0 The grey numbered circles in this user manual refers to the numbered information graphic and illustrations inside the cover

1 Get to know your TOPRO product

- | | |
|--|--|
| A Forearm support | H Adjustable handles (Premium) |
| B Release lever for height adjustment | I Hand brake (Premium) |
| C Lever for adjustment of forearm support | J Brake cable (Premium) |
| D Front wheels | K Lock wheel for the handle (Premium) |
| E Rear wheels | L Directional block (Premium) |
| F Foot brakes | M Product label |
| G Handles (Basic) | |

2 Safety

To give the right support and safety, walk straight and make sure the walker is kept as close to your body as possible (A). Walk between the rear wheels (B), not behind them (C).

Do not push the walker forward if the front wheels have stopped at an obstacle. The walker may tilt and cause an accident.

Do not use the walker on stairs. Be careful when passing thresholds etc.

Be aware that some parts of the walker might feel hot or cold when exposed to extreme temperatures.

Be aware not to put your fingers in open holes or between stationary parts.

Check the brakes before use every time.

Do not modify the product as this might put your safety at risk and the guarantee will become void.

When stationary or parked, make sure the parking brakes on both sides are activated.

Do not use the walker to transport heavy loads or persons.

Do not use the walker on slippery surfaces.

Keep the walker away from fire and hot items.

Do not climb on the walker or let children play with it.

3 Assembling and adjusting

TOPRO Taurus H is delivered with two levels of equipment:

Basic (814785) Adjustable arm pads and brakes on all wheels. Delivered partly assembled. Assembling: Please follow the steps in illustration 3.

Premium (814787) Has in addition to the basics adjustable handles and driving and parking brake on the handles. The rear wheels have a swing lock (directional block). Delivered completely assembled.

Before use, adjust the forearm supports and handles (see section 4).

4 Height adjustment / Forearm supports / Handles

Height adjusting: Pull the release levers (A) upwards. Use a little body weight to press down to decrease the height or reduce the counterweight to increase the height. Release the levers at the required height.

Recommended height: Stand with relaxed shoulders and position your elbows at a 90-degree angle (B). The level of the forearm supports should align to the level of your elbows. To help ensure a correct posture, a professional should help you with the adjustment of the forearm supports and the handles.

Adjusting the forearm supports: The forearm supports can be adjusted individually. Unscrew the lever under the forearm support (C) until the cushion is moveable. Adjust to the required position and tighten the screw. *NB! If the lever protrudes, it can be dangerous. Pull it down and rotate it away to be safely hidden.*

Adjusting the handles (TOPRO Taurus H Premium): Unscrew the lock wheel (D). Pull the handle (E) backwards or push it forwards and tilt it to the desired angle (F). After adjusting the handle, tighten the lock wheel (D). Be careful to not hurt your fingers in the process.

5 Foot brakes / Directional block

TOPRO Taurus H Basic – all wheels / TOPRO Taurus H Premium – front wheels: Step on the foot brakes on the rear wheels (A) and when required also on the front wheels (B), until you hear a clicking sound. The wheels are now in the locked position (C). To release them, step on the top of the foot brake pedal. TOPRO Taurus H Premium – rear wheels: Step on the foot brakes on the rear wheels (A) once until you hear a clicking sound. The wheels are now in the directional blocked position (E). Push the foot brakes further down until you hear a second click. The wheels are now locked (F). To release them, step on the top of the foot brake pedal (D).

6 Hand brakes (Taurus H Premium)

Ensure the brakes are working properly before every trip. *Do not drive or push the forearm walker while the parking brakes are activated or when the wheels are partly locked using the brakes. This indicates incorrect use and can cause wear and tear, which makes the tires and brake quality weaker.*

Parking brakes: Push both brake levers forwards (A) to activate the parking brakes. Be aware that the lever has some resistance before it stops and locks the rear wheel. Pull both brake levers towards you to release the parking brakes.

Driving brakes: Pull both brake levers towards you (B) to reduce speed. It is a two-handed braking system, which means that the brake is activated for the rear wheel on the same side of the walker as the brake lever. When applying only one of the brakes, the walker will turn to the same side when walking. Releasing the lever will stop the braking. *Do not continuously use the brakes when the walker is in motion.*

Adjusting / checking: Loosen the nut (C). Tighten / loosen the brake cable with the adjustment screw (D). Ensure that the brakes are working properly and only touching the wheels when in use. Adjust the brakes evenly on both sides.

Maintenance: Inspect the brakes regularly by following the information above. For best possible grip, wipe off the tires with a damp cloth. The wheels and brakes can be replaced if they are worn out or damaged.

Transport

The walker can be transported either in vertical or in horizontal position. It should be set in the lowest height position to save space and facilitate handling. Ensure that the foot brakes are activated. Be careful when loading the walker into a vehicle, giving attention to securing it.

Storage

The walker should be stored indoors and in an upright position. Do not place heavy items on top of it while stowed away.

Cleaning

The walker can be cleaned using a damp cloth. In case of mud and grime accumulation, a cloth with a mild washing detergent can be used followed by rinsing with lukewarm water. Do not use high pressure cleaners, abrasives or steam wash. Dry the walker using a soft cloth.

Troubleshooting

Brakes

Worn out / defect wheel	→	Replace the complete wheel
Not braking effectively	→	Adjust the brake cables
Worn out / defect brake mechanism on wheel	→	Replace the entire rear wheel section
Worn out / defect brake handle	→	Replace the complete brake handle
Worn out / damaged brake cable	→	Replace the brake cable

Height adjustment

The gas spring is worn out or damaged	→	Replace the gas spring
---------------------------------------	---	------------------------

Disinfection

Disinfection must only be carried out by authorized personnel. Please contact your local dealer or mobility aid centre. During disinfection sufficient protective equipment must be worn, and the instructions of the disinfectant manufacturer must be followed. The surface of the rollator should be cleaned with a soft cloth or sponge and an approved chemical disinfectant containing 70–80 % ethanol. We advise against disinfectant that contains chlorine or phenol. Do not use heat disinfection. *The manufacturer cannot be held responsible for any damage or injuries that might occur using harmful disinfectant/measures or disinfection carried out by unauthorized personnel.*

Inspection / Maintenance / Re-use

The wheel bearings are sealed and maintenance-free. The parts do not need to be lubricated. It is recommended that inspection and maintenance are carried out regularly (frequency is determined by how and how often the walker is used). Check the following items: Tubes, frame, screws, handles, forearm supports, brakes, wheels and accessories. This also applies when the walker is to be prepared for re-use. Please read detailed instructions about maintenance of brakes and cleaning/disinfection of the walker in this user manual.

Material / Recycling

The walker is made of plastic-coated aluminium tubes, plastic couplings, push handles of polyamide-6 and TPE, and forearm cushions made of polyurethane. Most parts can be recycled. Dispose the walker and its packaging according to the applicable regulations in your country. For information, please contact your public authorities.

Warranty

TOPRO Taurus H is guaranteed free for faults and defects for 2 years. Damage caused by incorrect use or parts that are exposed to natural wear and tear (e.g., brakes, wheels, and handles) are exempt from the 2-year warranty. For repairs during the warranty period, please contact your local mobility shop or dealer. The warranty is void if unauthorised spare parts or accessories have been or are being used on the product. The expected lifespan of the product is estimated to be 10 years if it is used in the correct manner according to this user manual and its safety and maintenance instructions.

7 Product label

- 1 - Gross weight
- 2 - Safe working load
- 3 - Maximum user weight
- 4 - Maximum height of walker
- 5 - Maximum width of walker
- 6 - Model name
- 7 - Model number
- 8 - Date of manufacture
- 9 - Serial number
- 10 - Global Trade Item Number
- 11 - CE marking (This product conforms to Medical Device Regulation (EU) 2017/745)
- 12 - GS1 DataMatrix
- 13 - Manufacturer
- 14 - Consult Instructions for use
- 15 - Intended for indoor use
- 16 - Caution! – Read Instruction for use
- 17 - Medical Device

Accessories

	TOPRO Item No.
One-handed brake	814026
Forearm side support, pair	814647
Tray	814810
Small basket	814821
Footplate	814645
Infusion stand	814793
Stand for body fluid boxes	814034
Oxygen bottle holder	814820

Accessories mounted on the walker may influence stability; we therefore advise you to use them with care. Accessories can be ordered separately to enable the forearm walker to be customised to individual needs. Contact your mobility shop, dealer or TOPRO for an updated overview of available accessories or visit our homepage topromobility.com.

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TOPRO Limited
TOPRO Mobility AB