**TOPRO
TAURUS H**

TOPRO

Made in Norway

NO Bruksanvisning 3–6
EN User manual 7–10
DE Bedienungsanleitung 11–14

www.topro.no

CE

A 814846

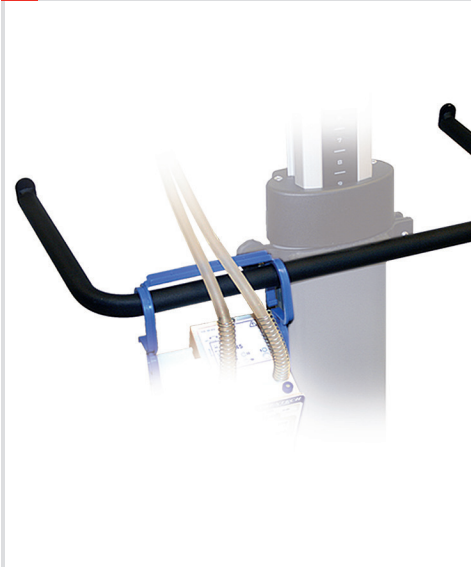
B 814001

C 814810

D 814034

E 814821

F 814820

G 814645

H 814793











I 814647

TOPRO TAURUS H

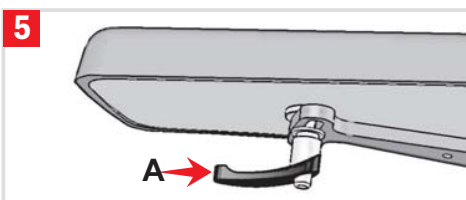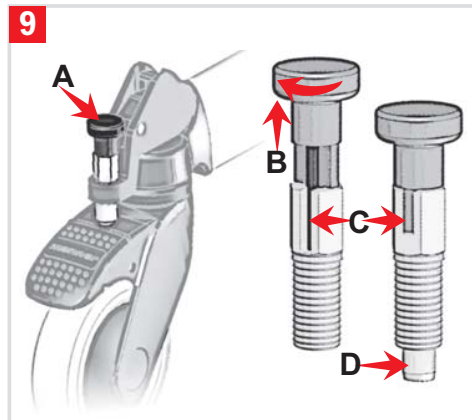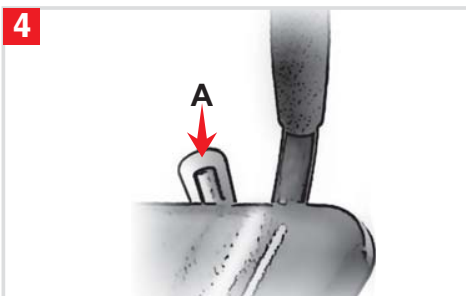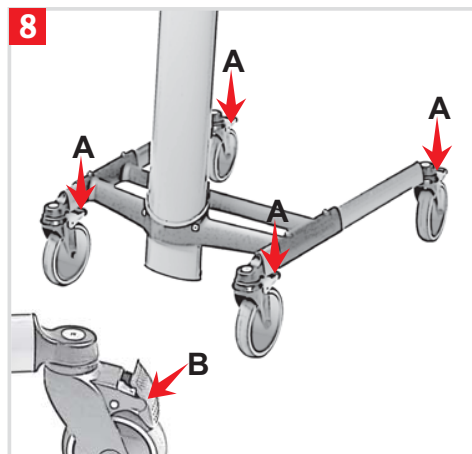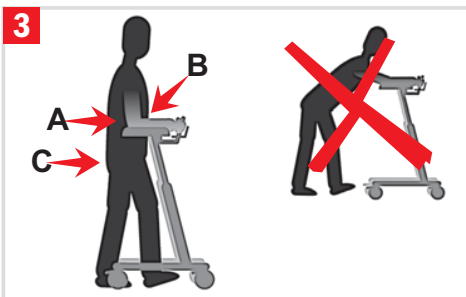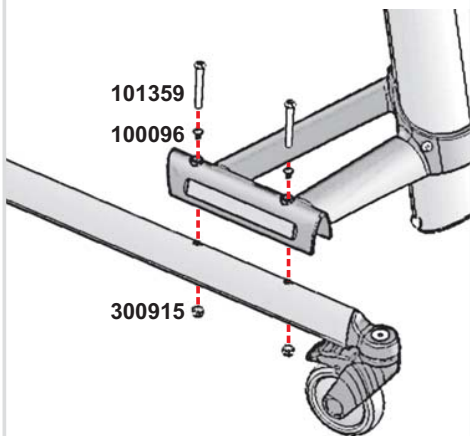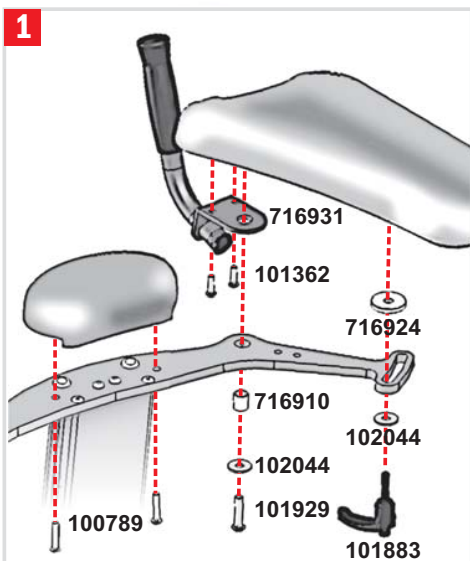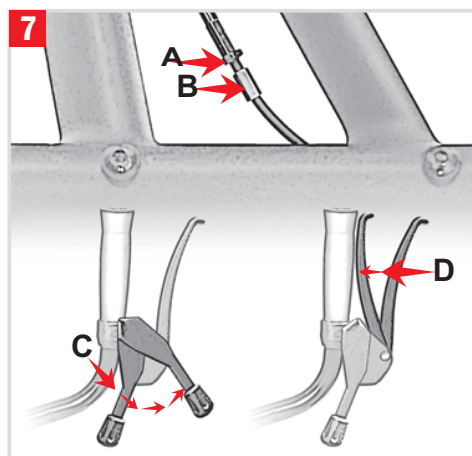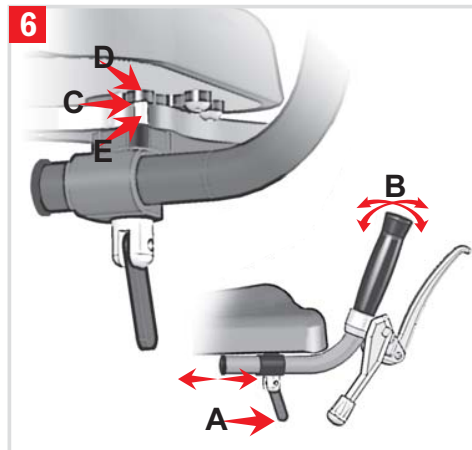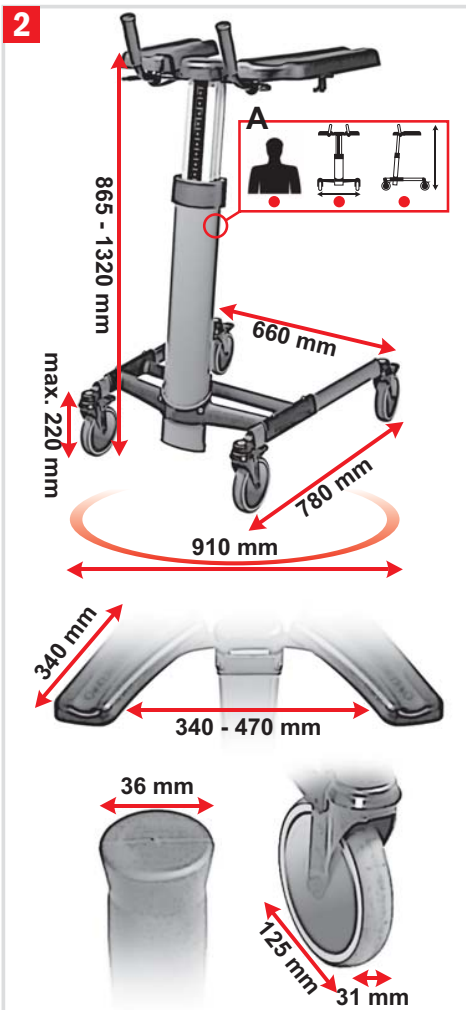BASIC 814785
PREMIUM 814787

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Welcome as a user of a TOPRO PRODUCT

Every TOPRO product is developed, designed and made in Norway. Advanced manufacturing equipment, quality and environmental security give great accuracy and maximum quality in all levels of production. The product is declared to be in conformity with the Medical Device Directive 93/42/EEC. It is tested and approved according to EN ISO 11199-3:2005.

On receipt of the goods, please check your product. If you have any queries please contact your mobility shop or dealer immediately. The same applies if you have general questions about the product. If you have problems reading this user manual, an electronic version is available on www.topro.no

Warranty // Expected life span of the product

TOPRO TAURUS is guaranteed free for faults and defects for 2 years. Damage caused by incorrect use, or parts that are exposed to natural wear and tear (such as e.g. brakes, wheels and handles) are exempt from the warranty. For repairs during the warranty period, please contact your local mobility shop or dealer. The warranty is void if unauthorised spare parts and accessories have been or are being used on the product.

The expected life span of the product is estimated to be 10 years, provided that it is used in the correct manner according to this user manual and its safety and maintenance instructions.

■ **The red numbered squares below refer to the numbered illustrations on the inside of the cover.**

Assembling and adjusting 1

TOPRO TAURUS H Basic need assembling. TOPRO TAURUS H Premium is delivered completely assembled. Adjust the forearm supports and handles, see figure 5 and 6.

Measurements 2

A The label on the rear side of the column shows the product name/model and

- Maximum weight of user
- Maximum width of walker
- Maximum height of walker

A separate label on the column indicates the serial number (SN) and production year and month.

Technical data

Maximum weight of user	150 kg
Recommended user height	140-210 cm
Height of forearm supports	865–1320 mm
Width between forearm supports	340–470 mm
Length forearm supports	340 mm
Maximum product length	780 mm
Maximum product width	660 mm
Diameter turning circle	910 mm
Diameter wheels	125 mm
Width wheels	31 mm
Diameter handles	36 mm
Grip distance brake levers	74 mm
Product weight	18 kg
Material	
Frame	Aluminium
Push handles	PVC
Brake handles	PA6
Forearm supports	PU-foam

TOPRO TAURUS H is delivered with two levels of equipment

Basic (814785) Adjustable arm pads and brakes on all wheels. Delivered partly assembled.

Premium (814787) In addition to adjustable arm pads and brakes on all wheels it has adjustable handles and a driving and parking brake on the handle. The rear wheels have a lock for swing (directional block). Delivered completely assembled.

Intention

The TOPRO TAURUS is a walker, which shall give support to users with reduced balance and/or reduced walking ability. The intended user is an adult. The walker is developed for pushing, not pulling.

The main user of the product is the patient him-/herself; however, the walker can also be operated by an assistant.

The walker is not suitable for persons with low strength in arms and legs, with very poor balance, or with considerable cognitive disabilities. The product is designed and approved for indoor use on plain foundation. We advise caution when using the TOPRO Taurus in private homes with narrow doorways, as the maximum width of the walker is 660 mm.

Important to know about safety and use **3**

- The walker must be adjusted to the recommended height for right support and safe walking: Stand with relaxed shoulders and have your elbow at a 90 degree angle **A**. Hold the walker as close to your body to maintain the correct support and a safe walk **B**. The easiest way to keep your balance is to walk straight, look forward and hold your backside in **C**.
- Do not push the walker too far from your body as this can cause unnecessary strain and you could fall.
- Do not push the walker forward if the front wheels have stopped at an obstacle. It may tilt and cause an accident.
- Do not use the walker on stairs. Be careful when descending thresholds etc.
- Maximum user weight is 150 kg. Maximum user height is 210 cm.
- Do not let children play with the walker.
- Some parts of the walker might feel very hot or cold when exposed to extreme temperatures.
- Do not put your fingers or any loose objects in the gap between the brake lever and the handle to avoid injury.
- Do not alter the product as this might put your safety at risk and the guarantee will not be valid.

Height adjustment **4**

Stepless hydraulic height adjustment

Pull the handle upwards **A**. Use a little body weight to press down or reduce the counter weight to increase the height. Release the handle at the required height.

Adjusting the forearm support **5**

The forearm supports are adjusted individually. Unscrew the lock underneath the forearm support **A** until the cushion is moveable. Place at the required position and tighten the screw. **NB!** If the lock sticks out, it can be dangerous. Pull the lever down and rotate it so that it does not stick out.

Angle adjustment and twisting the handle, TOPRO TAURUS H Premium **6**

Unscrew the screw **A** until you can move the handle. Adjust the handle **B** to the required angle, length and twist, and then tighten the screw. **NB!** The screw must not be screwed too far out as the parts will fall from each other. If this happens, then put the pin **C** in the holes **D** and **E**. It is not possible to pull the handles all the way out; there are physical stops both ways. See figure 1 for more detailed instructions.

Brakes, TOPRO TAURUS H Premium **7**

Adjusting the brakes:

Unscrew the nut **A**. Tighten / loosen the brake wire with the adjusting screw **B**. Ensure the brakes work properly and do not sweep when the brakes are off. Adjust the brakes evenly on both sides.

Important:

Ensure that the brakes work properly before each trip. Inspect the brakes frequently by following the information above. The walker wheels are made of a soft rubber, which gives an optimal braking capacity on slippery floor. Wrong use can cause unsafe situations and damage to the walker: *The walker must not be driven or pushed with force when the parking brakes are on or when the wheels are partly locked using the brakes.* This indicates incorrect use and can cause wear and tear, which makes the tires and brake quality weaker.

Using the parking brakes:

The brake lever **C** works on both rear wheels simultaneously. Push the brake lever forward. It has some resistance before it stops and locks both the rear wheels. Pull the brake lever backwards to release the rear wheels. Inspect the brakes frequently to make sure they work properly.

Using the driving brakes:

The brake lever **D** works on both rear wheels simultaneously. Pull the brake lever towards you to stop the rear wheels. When the brake lever is released, the rear wheels are released. Inspect the brakes frequently by following the information above.

Using the foot brakes **8**

Step on the foot brakes **A** on the rear wheels or when required on all wheels until you hear a clicking sound. The wheels are now in a locked position **B**. To release the wheels, step on the back of the foot brake pedal so that it comes up again.

Locking the swing (directional block), TOPRO TAURUS H Premium **9**

Locking the swing can be done with an index pin **A** on both wheels. Ensure the mounting bracket for the foot brake is pointing backwards.

This is how the index pin works:

Locking: Twist the upper part of the index pin **B** so that it fits into the track of the lower part **C**. By doing this the pin **D** will slide out and lock.

Releasing: Pull the upper part of the index pin **B** up and turn it 90 degrees. **NB!** Make sure it stays in the upper track. It will then lock in the upper position. The pin **D** slides in and the index pin is unlocked.

Please note: If there is no need to lock the swing but there is a risk that the user may unintentionally release the index pin, we recommend to remove it. This is done by screwing it out with an adjustable spanner.

Transport

The walker can be transported either in vertical or in horizontal position. The walker should be set in the lowest height position, to save space and ease the handling. Ensure that the foot brakes are on. Be careful when loading the walker into a vehicle, giving attention to securing of the item.

Storage

The walker should be stored indoors and in an upright position.

Material // Recycling

The walker is made of plastic coated aluminium pipe, plastic couplings and material of plastic and polyester. The forearm cushion is of polyurethane. Most parts can be recycled. Dispose the walker according to the applicable regulations in your country. For information, please contact your public authorities.

Cleaning

The walker can be hosed and washed with normal household detergents. Do not use scrub detergents, steam- or high-pressure cleaners.

Disinfection

Disinfection must only be carried out by authorized personnel and by wearing sufficient protective equipment. The surface must be cleaned with a disinfection that contains 70–80 % ethanol. We advise against disinfection containing chlorine and phenol. *The manufacturer cannot be held responsible for any damage or wounds that might occur by the use of harmful disinfection or disinfection carried out by unauthorized personnel.*

Inspection // Maintenance // Re-use

The wheel bearings are sealed and maintenance-free. The other parts do not need to be lubricated. It is recommended that general inspection and maintenance is carried out and that the items below are checked frequently (how frequent is determined by how and how often the walker is used): tubes, frame and screws, handles and forearm supports, brakes, wheels and accessories. This also applies when the walker is to be prepared for re-use. Please see detailed instructions in this user manual about maintenance of the brakes and cleaning and disinfection of the walker.

If the product does not work properly

1 = Possible reason

2 = Action

Brakes

1 Worn out / defect wheel

2 Change the complete wheel

1 Not braking effectively

2 Adjust the brake wires

1 Worn out / defect brake mechanism on wheel

2 Change the entire back wheel section

1 Worn out / defect brake handle

2 Change the complete brake handle

1 Worn out / damaged brake wire

2 Change the brake wire

Height adjustment

1 The gas spring is worn out or damaged

2 Change the gas spring

1 The height adjustment handle is maladjusted

2 Adjust the handle

Accessories

For pictures and TOPRO article numbers see the inside of the cover. Accessories mounted on the walker may influence stability, we therefore advise you to use them carefully.

- | | |
|--|--------------------------------------|
| A Brake handle left side (instead of right) | F Holder for oxygen |
| B Adjustable handles | G Footplate |
| C Table | H Infusion stand |
| D Stand for body fluid-boxes | I Forearm side support (pair) |
| E Small basket | |

The accessories are to be ordered separately to give the possibility to compose a TOPRO TAURUS that suits the individual users need. Contact your mobility shop, dealer or TOPRO for an updated overview of the accessories, or visit our homepage www.topro.co.uk

In case of questions or product faults, please contact your mobility shop or dealer.

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