

| Mobilising patients together

POZ 'IN' FORM **PLUS**

CAREWAVE **PLUS**

PATENT
REGISTERED



CE 

Class I medical devices



ECODESIGN



MADE IN
FRANCE

A new **PLUS** range

WIN  CARE



Repositioning

The most effective method for pressure ulcer prevention.

? Why?

- > Helps to offload and redistribute pressure
- > Ensures postural support
- > Corrects poor posture
- > Improves comfort and wellbeing
- > Aids caregivers with their tasks

who?






Anyone at risk of pressure damage, including individuals with neurological disorders, physical disabilities, and medical conditions which affect their mobility, functional or physical wellbeing

When? ¹

- > **Positional changes every 3 hours**
with a castellated foam mattress
- > **Positional changes every 4 hours**
with a static memory foam or multi-layer mattress
- > **Positional changes every 8 hours**
with a dynamic mattress or overlay in continuous low pressure or alternating pressure mode, or with low air loss

Note: (1) HAS - French health authorities guidelines

Must be used as part of a comprehensive package of care:

- +  Medical bed
- +  Pressure Relieving Surface
- +  Regular skin assessment
- +  Nutrition
- +  Appropriate seating solution = pressure relieving cushion (static or dynamic)



💡 Did you know?

There is a lack of access to positioning cushions in **French nursing homes**

70% Nursing homes are not equipped with positioning cushions

25% Only eligible patients have positioning cushions

Prolonged immobility leads to complications including pressure damage...
Introducing a new range of positioning cushions from Winncare



Developed in close cooperation with **clinicians**



Cover all **patients** needs



Repositioning with a **PLUS**

+ Accessible for health care professionals

+ Effective for patients

WIN  CARE

A new comprehensive range with a **PLUS**, suitable for the majority of patients across multiple care settings

Choose from 2 types of technology based on clinical need

POZ 'IN' FORM **PLUS**



microfibrils


For **POSITIONING**



Reduced mobility

> **COMFORT**

CAREWAVE **PLUS**

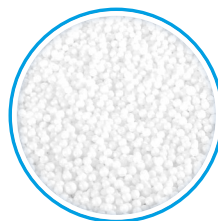

microbeads


For **POSTURE**



Reduced mobility to immobile

> **CARE**

17 shapes available dependent on patient need

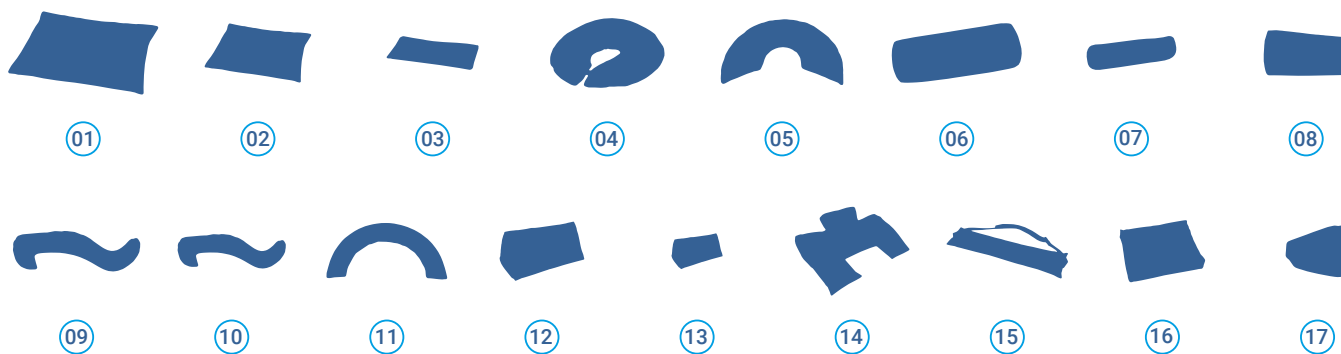

Choose from 3 cover materials bringing together **comfort, hygiene and durability** to meet the demands of any care settings

Standard cover



High density polyurethane

> **Easy to clean**



Comfort cover



Natural cellulose fibres

> A feeling of **comfort and security**



Hygiene cover



CAREWAVE **PLUS**

Polyurethane treated with silver ions ultrasonically welded

> **Safe and easy to use for multiple patients**



More than just a simple cover!

- ▶ Ag+ silver ion treatment, effective in preventing the proliferation of bacteria
- ▶ Ultrasonically welded, limiting fluid penetration
- ▶ **Memory Touch™** technology for effective immersion and pressure redistribution

All our materials have received class 1 **OEKO-TEX** certification the strictest level, indicating no harmful substances, suitable for the most sensitive skin.



01 & 02

CAREWAVE plus



POZ 'IN' FORM plus



Universal Cushion No.1 & No.2

Dimensions:

Cushion No.1: 45 x 30 x 10 cm

Cushion No.2: 55 x 40 x 12 cm

Polyvalent positioning cushions placed at the lower and/or upper limbs, depending on the patient's morphology, provide:

- ▶ Pressure relief in the weight-bearing regions
- ▶ Confidence that positioning is maintained throughout
- ▶ Protection of contact points and reduced friction at the malleoli, knees, and ankles



03

CAREWAVE plus



POZ 'IN' FORM plus



Universal Cushion No.3

Dimensions: 45 x 20 x 8 cm

Polyvalent positioning cushions placed at the lower and/or upper limbs, depending on the patient's morphology, provide:

- ▶ Pressure relief in the weight-bearing regions
- ▶ Confidence that positioning is maintained throughout
- ▶ Protection of contact points and reduced friction at the malleoli, knees, and ankles



04

CAREWAVE plus



POZ 'IN' FORM plus



Circular Cushion No.4

Dimensions: 50 x 45 x D10 cm

Positioning cushion, used with the cylindrical **No. 6** cushion for agitated patients, resulting in:

- ▶ The hip remaining abducted throughout
- ▶ Reduced of friction at the knees and protection of contact points



05

CAREWAVE plus



POZ 'IN' FORM plus



Semi-circular Cushion No.5

Dimensions: 60 x 35 x D10 cm

Positioning cushion, used with the cylindrical **No. 6** cushion for agitated patients, resulting in:

- ▶ The hip remaining abducted throughout
- ▶ Reduced of friction at the knees and protection of contact points



06

CAREWAVE plus



POZ 'IN' FORM plus



Cylindrical Cushion No.6

Dimensions: 60 x D20 cm

Polyvalent positioning cushion with several applications:

- ▶ When positioned under the knees, it reduces tension in the tendons and muscles at the lumbar spine, relaxing the abdominal wall.
- ▶ When placed under the calves, it relieves pressure at the ankles
- ▶ When used with a **No.4** circular cushion, it provides:
 - Protection of contact points and reduced friction at the malleoli, knees, and ankles
 - The lower limbs to remain in abduction



07

CAREWAVE plus



POZ 'IN 'FORM plus



Cylindrical Cushion No.7

Dimensions: 45 x D12 cm

Polyvalent positioning cushion with several applications:

- ▶ Protects contact points when placed between the malleoli, knees, or ankles, reducing friction



08

CAREWAVE plus



POZ 'IN 'FORM plus



Slanted Cushion No.8

Dimensions: 40 x 20 x 9 cm

Polyvalent positioning cushion with several applications:

- ▶ When placed at the lower and/or upper limbs in a patient who is not agitated, it provides:
 - Localised pressure relief
 - Assurance that the patient is supported in their chosen position



09 & 10

CAREWAVE plus



POZ 'IN 'FORM plus



Lateral 30° Positioning Cushion No.9 & No.10

Dimensions :

Coussin N°09 : 180 x 55 x 15 cm

Coussin N°10 : 170 x 55 x 15 cm

Polyvalent positioning cushion with several applications:

- ▶ When placed in a semi-lateral 30° position on the posterior surface of the patient, it relieves pressure in the following regions:
 - The trochanter, sacrum, ankles, internal spine and medial malleoli
- ▶ When placed in a semi-ventral position, it relieves pressure at the base of the lungs, clearing the upper airways
- ▶ When placed under the calves, it relieves pressure at the ankles
- ▶ When placed under the thighs, it reduces the tension in the tendons and muscles at the lumbar spine, relaxing the abdominal wall



11

CAREWAVE plus



POZ 'IN 'FORM plus



Half-moon Cushion No.11

Dimensions: 130 x 80 x 20 cm

Polyvalent positioning cushion with several applications:

- ▶ When placed behind the patient's chest, it allows the patient to assume a comfortable a semi-seated position with chest elevation
- ▶ When placed under the calves, it relieves pressure at the ankles
- ▶ When placed under the thighs, it reduces the tension in the tendons and muscles at the lumbar spine, relaxing the abdominal wall
- ▶ When used with a N°06 cylindrical cushion or a triangular No.12 cushion placed behind the knees, it allows the patient to assume the semi-fowler position



12

CAREWAVE plus



POZ 'IN 'FORM plus



Triangular Cushion No.12

Dimensions: 60 x 30 x 30 cm

Polyvalent positioning cushion with several applications:

- ▶ When placed under the calves, it relieves pressure at the ankles
- ▶ When placed along the back, and used with a No.7 cylindrical cushion or a No.3 universal cushion between the knees, it allows the patient to assume a semi-lateral 30° position



13

CAREWAVE plus



POZ 'IN' FORM plus



Small Triangular Cushion No.13

Dimensions: 45 x 25 x 25 cm

Polyvalent positioning cushion with several applications:

- ▶ When placed under the knees, it reduces the tension in the tendons and muscles at the lumbar spine
- ▶ When placed under the ankles, it relieves pressure locally

**14**

CAREWAVE plus



POZ 'IN' FORM plus



Semi-fowler Cushion No. 14

Dimensions: 70 x 50 x 10 cm

Positioning cushion for a semi-seated position

- ▶ When placed behind the chest, it provides bilateral support of the chest
- ▶ When used with a **No.12** triangular cushion or a **No.6** cylindrical cushion placed behind the knees, it provides the following:
 - Pressure relief at the diaphragm, facilitating respiration and oxygenation
 - A reduction of contact pressure at the sacrum, lower back and buttocks
 - Prevention of gastroesophageal reflux

**15**

CAREWAVE plus



POZ 'IN' FORM plus



Heel off-loading Cushion No.15

Dimensions: 80 x 20 x D15 cm

Positioning cushion for heel rest, provides the following:

- ▶ When placed under the ankles, it provides offloading of the heels
- ▶ When used with 1 or 2 **No.7** cushions placed behind the knee(s), it reduces the tension in the tendons and muscles at the lumbar spine

**16**

POZ 'IN' FORM plus



Occipital Rest Cushion No.16

Dimensions: 35 x 28 cm

Positioning cushion for occipital rest; when placed under the head it provides:

- ▶ Stable head positioning
- ▶ Pressure relief at the occipital region
- ▶ Pressure relief for the ear in the lateral position

**17**

CAREWAVE plus



POZ 'IN' FORM plus



Thigh Abduction Cushion No.17

Dimensions: 35 x 28 cm

Polyvalent positioning cushion to be placed between the legs, supports:

- ▶ The hip to remain abducted throughout
- ▶ Reduced friction and improved comfort between the thighs and knees



With intuitive use & easy access tools

Making re-positioning more accessible

Numbers associated to each shape:

- > Fast identification
- > Associations between shapes easier

QR codes on each cover for quick and easy access to:

- > key product information

Intuitive tools:

- > Usage tips that are readily available via QR codes
- > Simple brochures that can be understood by everyone
- > Posters showing key positions
- > Videos sharing best practice, with advice and tips



A sustainable range with a **PLUS**, placing **ecodesign** at the heart of our innovation



When designing our new range, we thought about how to minimise its environmental impact throughout its lifetime.



Products that are carefully **designed and manufactured locally** at our French sites, adhering to **rigorous environmental regulations**



Recyclable packaging, made up of **over 30% recycled material**



Natural high quality, longer lasting and more environmentally friendly.

Class 1 **OEKO-TEX** certified - the strictest level, indicating no harmful substances, suitable for the most sensitive skin



100% natural colour undyed threads used for the covers.

This saves water, in addition to chemicals and energy used in dying processes

Technical Characteristics

Name	Shape	Size in cm (L x D x H or Diam)		COVER			
				PROMUST PUHD	Lenzing FR	PROMUST C.I.D	
		CAREWAVE PLUS	POZ'N'FORM PLUS				
Universal Cushion	01		45 x 30 x 10	45 x 30 x 15	•	•	•
	02		55 x 40 x 12	55 x 40 x 15	•	•	•
	03		45 x 20 x 8	45 x 20 x 10	•	•	•
Circular Cushion	04		50 x 45 x D10	50 x 45 x D15	•	•	•
Semi-circular Cushion	05		60 x 35 x D10	60 x 35 x D15	•	•	•
Cylindrical Cushion	06		60 x D20		•	•	•
	07		45 x D12		•	•	•
Slanted Cushion	08		40 x 20 x 9	40 x 20 x 12	•	•	•
lateral 30° positioning Cushion	09		180 x 55 x 15		•	•	•
	10		170 x 55 x 15		•	•	•
Half-moon Cushion	11		130 x 80 x 20		•	•	•
Triangular Cushion	12		60 x 30 x 30		•	•	•
Small Triangular Cushion	13		45 x 25 x 25		•	•	•
Semi-fowler Cushion	14		70 x 50 x 10		•	•	•
Heel off-loading Cushion	15		80 x 20 x D15		•	•	-
Occipital Rest Cushion	16		-	50 x 40 x D10	•	•	-
Thigh Abduction Cushion	17		38 x 30 x 12	38 x 30 x 16	•	•	•

Reference

POZ 'IN' FORM plus

Reference template:
VPOZ[Cushion number][Cover]

SG (Cover)
Standard Grey

CG (Cover)
Comfort Grey

Example:
 Cushion
 Triangular No.12
 Cover
 Standard Grey
 Reference: VPOZ12SG

CAREWAVE plus

Reference template for the cushion with
the Standard Blue and Hygiene Blue covers:
VCAR[Cushion No.][Cover]

SB (Cover)
Standard Blue

HB (Cover)
Hygiene Blue

Example:
 Cushion
 half-moon No.11
 Cover
 Standard Blue
 Reference: VCAR11SB

Reference template for cover
 Comfort blue sold separately:
 VHCAR[Cushion No.][CB]

CB (for cover)
Comfort Blue

Example:
 For cover
 Comfort Blue
 for cover
 half-moon No.11
 Reference: VHCAR11CB

WINNCARE France
4, Le Pas du château, 85670 Saint-Paul-Mont-Penit (France)

Production site
Miniac-Morvan (35)



Class I medical devices



WIN CARE

winnicare.com

